

Jive!

with
IDV Life
Inside



Getting Started

This guide is not a replacement for the manual or structured training. It is intended to allow you to get an overview of using the Jive!.

We hope that you find this guide useful and that your first impressions of the Jive! are positive as a result. If you have any questions do not hesitate to contact us. We wish you many happy years of use. The Jive! is a purpose built Voice Output Communication Aid. At Possum, we have a long history of supplying solid, reliable devices to the disability market. We understand the issues associated with the daily experience of our system users. These have been considered at every stage of the design process.

The QuickStart guide is designed to give a brief overview of Hardware, Software and Vocabularies and do not replace a proper training session carried out by a Possum trainer. The manual is available on the enclosed DVD. As a dedicated communication aid, The Possum Jive! will remain an extremely reliable and easy to use communication aid for years to come.



Ready to

Jive!?

Headphone connector



Power in


USB

Getting Started

- 1) Remove the Jive unit from packaging and plug in the mains power adapter to both the mains and the Jive!
- 2) Press, hold for 1 second and release the Power button.

Reset

Power button

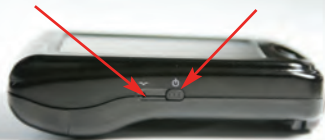


Infra-Red (IR) out

Microphone



Speaker

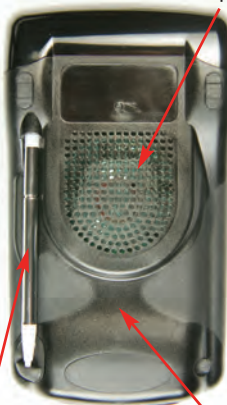


Stylus

Infra-Red (IR) in



Mains power adapter

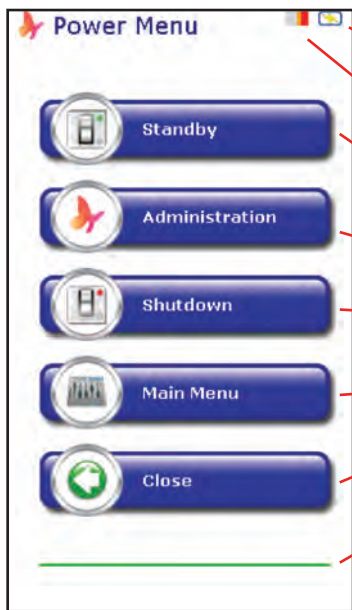

Power Management

- Mains indicator
- Battery indicator
- Put the Jive! into standby
- Enter unlocked administration
- Shutdown the Jive!
- Go to Main Menu
- Close this Window
- Auto close of window via timer

Turning On your Jive! Press, hold for 1 second and release the Power button.

Turning off the Jive! for storage. With the unit turned on, press and hold the power button and select shutdown.

Going into standby

1) Allow the power menu settings to take effect and your Jive! will go into standby automatically.

Or:

2) Press,, hold for 1 second and release the power button and select standby.

Charging your Jive! Attach your Red and Black Possum HC501J charger to the 'Power in' at the bottom of the unit. We recommend daily overnight charging for your unit. A full charge should take about six hours. **Battery and mains status** In the top right hand corner of the Main Menu, Power menu and Administration screens, there is a battery status indicator and a mains attached indicator. The battery status indicator works on a traffic light basis where green is good and red is bad. The battery status indicator consists of six bars: Green = charged, Red = low.

Battery isolate and reset In the unlikely event you need to reset your Jive!

1) Remove any Power or USB connections from the Jive!

2) Insert the stylus into the Reset aperture on the left hand edge of the Jive! for a few seconds until the screen appears blank, and then release it.

3) The unit will then restart. You may want to attach the mains at this time.

The Jive! can only run it's own software.

When you first turn your Jive! on using the power button on the left hand edge, you will be presented with the "Home page" of the sample layout. This is designed as a snapshot of the capabilities of the Jive! and is not intended to be used for communication. You should load one of the preset users on the device.

Vocabulary QuickStart

The first time you turn the Jive on you will be presented with a Home page which will allow you to explore the various levels of communication available to you. We are proud to present the first ever layouts which merge the communication worlds of Conversational, Curriculum, Sentence Building and Environmental Control vocabulary - "IDV4life".

They are set at 5 levels:

IDV4Life-1 - a simple introductory vocabulary of 3x3 pages with no user navigation required

IDV4Life-2 - a simple introductory vocabulary of 4x3 pages with user guided navigation introduced

IDV4LifeA - entry level sentence building


IDV4LifeB - 3 word sentence building, increased navigation

IDV4LifeC - more complex sentence building, 3 level navigation



Using Jive!

At this stage, you can experiment with the sample pages provided and determine at what level you would like to work with the user of the Jive!. It is extremely important that you do this now as it may save a lot of time and effort later on when you have customised a layout specific to your user.

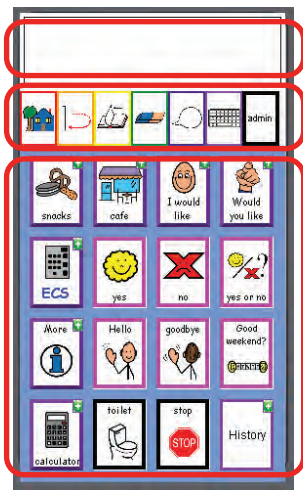

Information Bar (IB)

Messages are built here. Selecting the IB will cause the Jive! to speak. Selecting an empty IB will recall the last message spoken.

Command Bar (CB)

Universal commands and messages are accessed from the CB. You can edit the CB and dynamically load different CB. See centre spread for more information.

Layout (LO)

A Layout is a collection of pages containing cells. Cells can navigate to pages, build messages, send IR and a lot more besides.

Two Screen Editing (A&B)

- 1) Press and release the power button, tap 'Administration'.
- 2) Type in code 5483 and then tap the green tick.
- 3) After a few seconds 'Page Central' will appear - for 'Page Central' features see figure 'A' overleaf.
- 4) Tap a cell to edit.
- 5) "Cell Edit" screen will appear - for 'Cell Edit' features see figure 'B' overleaf.
- 6) Tap 'Add Symbol'.
- 7) This will open the 'Symbol Picker' screen where you can search for a symbol and additionally using text auto fill assign a label, message and auditory prompt.
- 8) Type in the search term 'Snacks' and tap the 'torch'.
- 9) Tap the 'Snacks' image.
- 10) Select the 'Text Auto Fill' to fill the message label.
- 11) You can change the message label text in the 'Use this Text' box. Please place a space after the text.
- 12) Tap the green tick.
- 13) Set 'Message Type' - see figure 'C' overleaf.
- 14) Tap the 🗨️. The 'Save Changes' box will appear, tap 'Yes' and tap 'OK'. You are now in 'user mode'.

Page Link - Adds a page to the symbol


- 1) Press and hold the power button, tap 'Administration'.
- 2) Type in code 5483 and then tap the green tick.
- 3) After a few seconds "Page Central" will appear -for 'Page Central' features see figure 'B' overleaf.
- 4) Tap a cell to edit.
- 5) "Cell Edit" screen will appear -for 'Cell Edit' features see figure 'B' overleaf.
- 6) Tap the 'Add Page' icon.

Then either:

- A) Select an existing page to navigate to from the drop down list and tap the green tick to activate.
- or B) Select 'New Page' from the drop down list
> Enter the Page Name > Create > Yes > OK > Tap the Green Tick
> Tap the 🗨️. The 'Save Changes' box will appear, select 'Yes' and select 'OK'. You are now in 'user mode'.



Command Bar

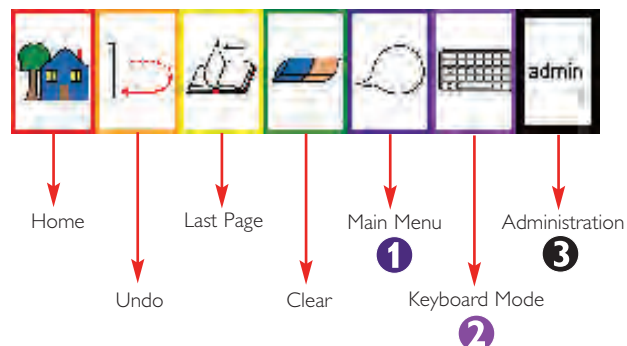

1 MAIN MENU

- Volume up or down
- Sound System beeps on or off
- Speak Speech on or off
- Zoom Zoom messages to full screen on or off
- Private Private (non-Learning) on or off
- Close Close this window

2 KEYBOARD MODE



Word Prediction Area
2, 4, 6 or 8 word predictions can be displayed here. If sentence prediction is enabled any sentences available after a word is chosen will appear here.

Keyboard
Various keyboards and number pads can be displayed here. Colours and fonts can be varied. If character prediction is enabled then demoted characters will be a different colour.



Sentence Prediction Area
When sentence prediction is enabled (via administration selecting 'text' then 'prediction') then available sentences will be shown after selecting a word. Typing another character will bring back word prediction

Keyboard
Various keyboards and number pads can be displayed here. In this case a qwerty layout is being shown.

3 ADMINISTRATION

Entering Administration

- Method 1**
- 1) Press the power button
 - 2) Select administration
 - 3) Enter the PIN (default = 5483)
 - 4) Select the green tick
 - 5) After a few seconds you will be in Page Central with edit protection unlocked

- Method 2**
- 1) If your command bar has an Admin button select that
 - 2) After a few seconds you will be at the administration top level in Edit protection mode



Tap the butterfly in the top left hand corner to return to the Administration menu at any time.

Design **i**

Layout Add, edit, delete and load layouts

Page Central Set up pages and their cells

Page Manager Add, edit, delete and load pages

Command Bar Set up command bars and their cells

Command Bar Manager Add, edit, delete and load command bars

Experience **ii**

Appearance Change the screen layout and zoom

Emphasis Set up highlighting, beeps and magnification

Speech Choose voices and feedback methods

Pronunciations Change the way words are said

ACCESS Change the access method

Keyboard Keyboard colours and layout

Text **iii**

Prediction Character, word and sentence prediction

Words Maintain learnt words

Sentences Maintain learnt sentences

Abbreviations Abbreviations, irregular verbs and plurals

Symbol **iv**

Symbol Manager Remove and add custom symbols (PC only)

Theme Manager Setup cell themes for consistent appearance

Management **v**

Audio Manager Record, play and delete sounds and import (PC)

Remote Control Learn IR and maintain devices (Jive! only)

Favourite Appliances Map appliances to shortcuts

Configuration **vi**

Preferences Choose what is loaded when the device starts


Start Up Check start up options

Edit Protection Lock down parts of the Jive! Talk software

Power Set warnings and power down times

About Find out software and library information

Tool Bars

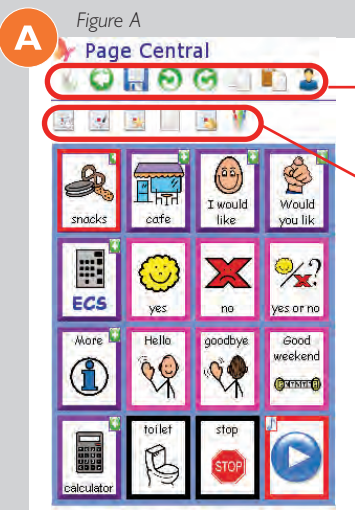

Primary Tool Bar
Secondary Tool Bar

A1 PRIMARY TOOL BAR

- Last Screen
- Undo
- Copy
- User Mode
- Reset To Default
- Save
- Redo
- Paste

A2 SECONDARY TOOL BAR

- Edit
- Select
- Swap
- Page Manager
- Themes

Edit: go to cell edit screen and edit selected cells. If in single cell mode just tap the cell to edit directly

Select: choose how many cells to edit at once

Swap: tap to turn on. Drag and drop cells onto one another to swap them. Tap to turn off

Page Manager: load, edit and create pages

Page Setup: see menu below

Themes: assign themes to one or more cells



Cell Edit Tool Bars

B2 MESSAGE TYPE

Message Type	Definition
Add Message	The contents of the message field are added to the information bar.
Add Message and Speak	The contents of the message field are added to the information bar and the full contents of the information bar are spoken.
Clear and Add Message	The information bar is cleared and then the contents of the message field are added to the information bar.
Clear, Add Message and Speak	The information bar is cleared and then the contents of the message field are added to the information bar and the full contents of the information bar are spoken. This sentence will then be 'learnt' for future use.
None	Cell will not speak – generally used with an advanced action or page link.
Speak Message	The message text is spoken – the sentence will not be learnt.

B3 CELL ACTIONS

- Cell contains a sound
- Cell links to another page
- Cell has an action
- Cell is a scrolling cell

B1 CELL EDIT TOOL BARS

- Move from cell to cell. Hold an arrow to see where you are on the page
- Demote, promote and add scrolling cells
- Clear cell contents

- Add symbol
- Label text, font, size, colour and position
- Auditory prompt
- Add an action to a cell
- Cell border thickness and shape
- Cell border colour
- Cell background colour
- Tense indicator overlay on symbol
- Modifier indicator overlay on symbol
- Add sound to cell
- Link to another page
- Apply theme to cell

Approvals

The Possum HC7000 Jive! is CE marked under the EMC directive and has been tested to EN55022:2006 (EMC)
EN55024:1998, Amd. 1 & Amd. 2:2003 (EMC)

Servicing

There are no user-serviceable parts within the HC7000. Refer all faults to Possum Limited or their authorised service agents.

Warning! Hazardous voltages and exposed live parts exist within the HC501J power supply unit when in use. Never attempt to open the unit or remove the case cover.

Power Requirements

The Possum HC7000 is designed to operate only with the supplied mains power adapter - type HC501J. (Never use any other power supply)

Operating / Storage Environment

Storage Temperature Range: 0°C to 40°C
(32°F to 104°F)

Storage Humidity Range: 30% to 75%
(non-condensing)

Storage Altitude Range: 0m to 2,000m

Physical Characteristics

Width: 85mm

Length: 145mm

Depth: 40mm

Weight: 350grammes




possum
helping independence emerge

Possum Ltd
8 Farnbrough Close
Stocklake Park Industrial Estate
Aylesbury
Buckinghamshire
United Kingdom
HP20 1DQ
Phone: +44 (0) 1296 461000
Fax: +44 (0) 1296 461001
E-mail: jive@possum.co.uk
Web: www.possum.co.uk